12.75"

accordion fold to: 4.25"

Silicone
ONE-PIECE
Breast Pump

Easy To Use

Helpful Tips

Squeeze the base of the pump to create suction.

Position the flange over your breast, centering the nipple in the neck of the pump.

Release the base, creating suction on nipple.

Gentle, constant suction will draw milk out and keep pump in place - no hands needed!

Lift edge of flange to remove, and immediately transfer milk to storage bottle or other container.

CARE & CLEANING:
Clean and sterilize before first use and after each use. Dishwasher safe, or simply hand wash in warm, soapy water with sponge or soft bristled brush.
Rinse & dry thoroughly. Safe to sterilize in boiling water, or steam and microwave sterilizers. After sterilizing, let cool before using.
Do not use bleach, abrasive cleaning agents or harsh chemicals.
Rough scrubbing can damage surface. Store in a cool dry place away from sunlight.
Inspect before each use, checking for tears or breaks. If damage is found, discard & replace.
CAUTION:
Using a breast pump should be a comfortable experience. Discontinue use if pain occurs and persists after adjustments to placement are made.
Consult your healthcare provider or a lactation consultant if you have any questions regarding breastfeeding & pumping.

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©2020 Handi-Craft Company. All Rights Reserved.
Dr. Brown's and the drbrownbaby.com are trademarks of Handi-Craft Company.
Dr. Brown's and the drbrownbaby.com are registered trademarks of Handi-Craft Company.